


## 6.2.2 Setting Test phase end time of Step/Walk test


If the end test phase is not manually reviewed, the time exported in the file header will be where the software algorithm has guessed the end of the test to be. Be aware that for a standard 8 min step test, the software often estimates test duration to be 8.3 min. It is advisable to set the end of test phase in the commercial software but this may also be done in the academic software.

- Right click on the bottom screen. This will toggle between accelerometry waveform only, ECG only, or both. It is easiest to set end of test phase if just the accelerometry waveform trace is viewed.
- Scroll along on the trace to show waveform trace of the area where the test finished.

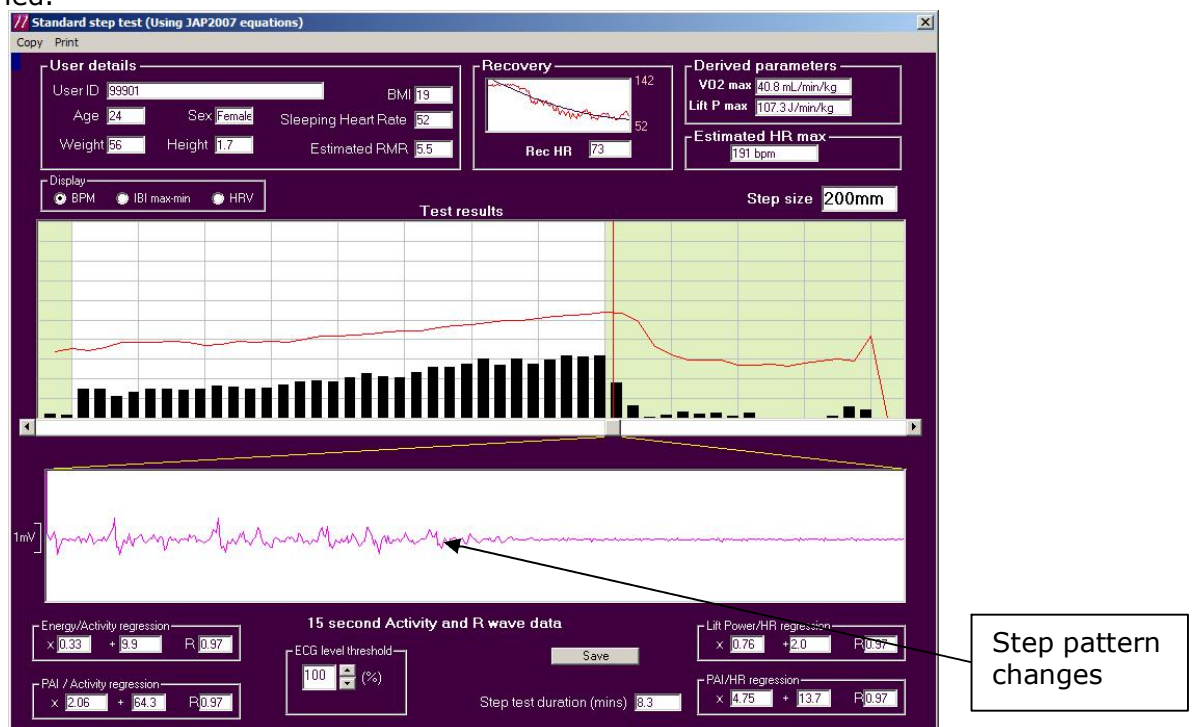

- It can easily be seen from the trace where the movement pattern stopped. Point at this place with the cursor and double click.

The following screen will appear:

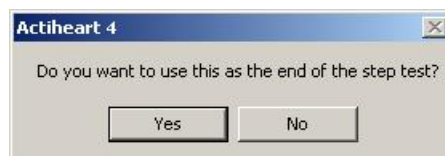

- Click 'Yes' when you are sure that the end point has been placed in the correct place
- The red line on the upper screen should now have moved to the end of the step test.
- When the file is exported, the end phase time will now appear in the header to be used for annotation.